

Home Irrigator Kit

A convenient user-friendly 1.5 litre Home Irrigator bag plus accessories

- 1 x flexible pipe
- 1 x clip
- 1 x short cannula (rectal nozzle)



Quantity:

1 x 1.5 litre bag

Type:

Pouch

Code:

CS003

Directions for use:

1. Pipe assembly

Take the cannula and insert into the pipe of the plastic irrigator bag, push on firmly to avoid leaks.

2. Filling

Check that the clip is properly secured to stem the fluid flow. Pour the required fluid into the irrigator bag. The fluid needs to fill the entire pipe to prevent air locks during the enema. To remove the air, open the clip and drain some fluid through the pipe then re-secure the clip. The pressure of the fluid flow is dependent on the height between the cannula and the irrigator bag, so always keep the irrigator bag positioned above the level of the clip to avoid backflow.

3. Application

Identify a convenient position to place the enema kit and secure the enema kit in place. Then laydown in a comfortable position and insert the cannula, release the clip to release the fluid and complete the enema process.

4. After use

We recommend this product for personal use. After using, clean with a disinfectant or warm soapy water (not boiling). Do not hesitate to remove and clean all parts used including the inside of the cannula and pipe.

Keep dry and store away from direct heat.

Keep out of the reach of children.



WE DO NOT INVEST
IN ANIMAL TRIALS



UK LABEL COMPLIANCE
REVIEWED